

Uses

The CoolTone™ device is FDA-cleared for improvement of abdominal tone, strengthening of the abdominal muscles, and development for firmer abdomen. CoolTone™ is also FDA-cleared for strengthening, toning, and firming of buttocks and thighs.

Important Safety Information

The CoolTone™ procedure is not for everyone. You should not have the CoolTone™ treatment in areas with metal, electrical, or electronic implants/devices like cardiac pacemakers, implanted hearing devices, implanted defibrillators, implanted neurostimulators, drug pumps, or hearing aids.

Tell your doctor if you have any medical conditions as CoolTone™ should not be used over a menstruating uterus, over areas of the skin that lack normal sensation, in patients with fever, malignant tumor, hemorrhagic conditions, epilepsy, recent surgical procedure, pulmonary insufficiency, or pregnancy.

CoolTone™ should be used with caution in patients with Graves' disease (an autoimmune disorder that causes overactive thyroid), active bleeding disorders, or seizure disorders.

Women who are close to menstruation may find that it comes sooner, or cramping is increased or intensified with CoolTone™ treatments, therefore, it is recommended to not undergo treatment during this time of the month.

CoolTone™ should not be used in the heart or head areas, areas of new bone growth, over the carotid sinus nerves, or over the neck or mouth. CoolTone™ should not be applied over swollen, infected, inflamed areas or skin eruptions. Caution should be used for patients with suspected or diagnosed heart problems.

Common side effects may include, but may not be limited to, muscular pain, temporary muscle spasm, temporary joint or tendon pain, and redness at or near the treatment site.

Ask your Healthcare Provider if CoolTone™ is right for you.